# SafeDrop™ Sharps Mail Back Program

A convenient service to dispose of your sharps waste

## Your Container System Includes:

Container size may vary

- This instruction sheet
- White return shipping box with biohazard symbol and Merchandise Return Label
- Brown inner box
- Plastic bag with tie
- Red sharps collection container with absorbent material inside
- Tape strips
- Serialized tracking document in a plastic pouch on return shipping box

This SafeDrop container system contains supplies you need to help you safely package and dispose of your sharps waste. When properly packaged by the waste generator, this SafeDrop container system complies with U.S. Postal Service requirements for shipping sharps waste for disposal through the U.S. Mail. **DO NOT** place this container system in the mail if it has not been properly assembled and secured. **DO NOT EXCEED THE MAXIMUM RETURN WEIGHT LISTED ON THE WHITE BOX.**

## GETTING STARTED

### Save the System Components:

1. Remove the white kit from the mailing box you received it in. **DO NOT** discard the white box or its contents. They will be needed to return your full SafeDrop container system for disposal.
2. Open the white box and the brown inner box, but **DO NOT REMOVE** brown inner box from white box.
3. Open the plastic bag inside the brown inner box and remove red sharps container.
4. **DO NOT** discard the white return shipping box, or any of the contents! Save all of these packaging items in a dry area. They must be used to properly reassemble the container system prior to mailing.

### Place Used Sharps into the Sharps Container:

Place the red sharps container near the point of use. Always be sure to keep the container away from children.

1. Place sharps into container with sharp end pointed away from you.
2. Once the contents reach the maximum fill line, snap the cap onto the base and press down to secure. Make sure the cap is firmly affixed prior to shipping.

### ⊗ These items **DON'T** go in the SafeDrop sharps container:

- Human remains
- Pathological waste
- Category A waste
- Hazardous waste
- Radioactive waste
- Pharmaceutical waste (including controlled substances)
- Chemical wastes
- Universal wastes
- Compressed gas cylinders, canisters, inhalers or aerosol cans
- Chemotherapy waste

Scan the QR code for Stericycle's Waste Acceptance Policy and complete list of prohibited items.

## IMPORTANT REMINDERS:

- Fill the sharps container with sharps waste only (used injector pens, syringes, lancets, etc.)
- **DO NOT OVERFILL!**
- Total residual fluid allowed is limited to 50 ml

*Sharps Medical Waste, for USPS purposes, means "a medical waste object that is capable of cutting or penetrating skin or packaging material and that is contaminated with a pathogen or may become contaminated with a pathogen derived from the medical treatment, diagnosis, immunization, or biomedical research of a human or animal. Sharps include used medical waste such as needles, syringes, scalpels, broken glass, culture slides, culture dishes, broken capillary tubes, broken rigid plastic, and exposed ends of dental wires."*

Reference: USPS Publication 52

# Preparing the SafeDrop Container System for Return Shipment

Once the red container is full, you're ready to return your postage-paid container system

## Seal the Container System

Once ready for final closure, close the sharps container. Carefully place the sealed red sharps container into the plastic bag. Securely seal the plastic bag using the tie provided.

## Prepare for Mailing

Insert sealed red sharps container into the return box and seal brown inner box shut with provided tape strip.

## Keep a Record

Remove serialized tracking document from pouch on side of white box. Do not damage pouch; completed form must be reinserted and pouch zipped shut.

## Mail the Container System

Close white outer shipping box and tape shut with provided strips. Use extra tape if necessary. Enter your complete return address on the Merchandise Return Service mailing label on top of box.

**NO P.O. BOXES ALLOWED.**

Mail the properly sealed kit through a U.S. Postal Office or give to your postal carrier for mailing.

- In the section marked "Generator," where your address is printed, **VERIFY YOUR INFORMATION IS CORRECT** or if not, print or correct information. **YOU MUST INCLUDE A VALID PHONE NUMBER.** In the section marked, "Generator Certification", print and sign your name.
- Remove bottom copy of document and keep for your records.
- Place three remaining copies back into pouch and close.

If you have any questions or are missing any part of this SafeDrop container system, please contact us at **866-783-3155** or visit **stericycle.com**

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